EN

Content

Water, degradable starch microspheres (DSM), sodium chloride, bromothymol blue (BTB), phosphate buffer.

Description

SmartPAN Biochemical Leakage Indicator is a sterile hydrogel containing degradable starch microspheres loaded with bromothymol blue. It is provided as a single-use plastic applicator packaged in a blister pouch. SmartPAN comes ready-to-use and can be applied evenly to the surgical field. SmartPAN is intended to be used during every surgical intervention at the pancreas including minimally invasive surgery. SmartPAN enables the surgeon to localize pancreatic leakage, to evaluate quantity, and to perform refined closure. Visualization and targeted closure of leakage from pancreatic tissue can decrease occurrence of postoperative biochemical leakage which can lead to pancreatic fistula being one of the most important morbidity issues in pancreatic surgery.

Medical Devices Directive (MDD)

SmartPAN complies with the Medical Devices Directive MDD 93/42/EEC and carries the CE mark.

Intended use / therapeutic indications

SmartIndicators are biodegradable indicators intended for visual detection of enzymatic leakage after tissue closure during surgical intervention. SmartPAN is indicated for visual detection of biochemical leakage from the pancreas after pancreatic tissue closure during surgical intervention.

Contraindications

- Do not use on patients with a known hypersensitivity to bromothymol blue.
- Do not use on patients with known allergy to potato starch.

Potential side effects

To date there are no implications known such as side effects or local or systemic toxicity of SmartPAN usage. Studies have shown almost no absorption of SmartPAN or its degradation products into the bloodstream. Nevertheless, rinsing of the surgical site after usage and drainage or suctioning of jet fluid should be assured to avoid accumulation of the product in the abdominal cavity.

- Incorrect usage of SmartPAN may result in false negative indication and thereby leaving a biochemical leakage unattended.

Interactions

To date there are no known interactions of SmartPAN such as interference with other perioperative medical actions or procedures. Nevertheless, usage should be accompanied by continuous monitoring of vital functions as regularly performed during abdominal surgery.

Caution

- For the safe use of SmartPAN the preparation and administration have to be done by health care professionals experienced with pancreatic surgery.
- Extensive usage of thermoelectric hemostatic devices may alter tissue color and thereby impair visualization of SmartPAN indication.
- Application of SmartPAN needs to be conducted carefully to place an even product layer on the tissue surface of interest without relevant time-delay during application. Optimal product layer covers tissue surface completely and homogeneously without local heterogeneous increase of layer-thickness.
- Syringe-style applicator allows for standardized usage to assure precise and comparable indicator reaction. SmartPAN appears orange when applied, but locally turns green-bluish over pancreatic leakage. This indicator reaction happens immediately or within seconds. Color change only appears in close proximity to relevant pancreatic leakage.
- SmartPAN should be used immediately after removal of the cap to avoid contamination of the product.
- SmartPAN is packaged for single patient use only. Discard remaining opened material after each medical procedure.
- SmartPAN is a sterile hydrogel. Do not re-sterilize SmartPAN.
- Do not place SmartPAN in a procedure pack/tray that will be sterilized prior to use. Exposure of SmartPAN to excessive heat (as in autoclaves or ethylene oxide sterilization) or radiation (such as gamma or electron beam) will alter its viscosity and may render the product unusable.
- Do not use more than one device per treatment.

Miscellaneous

- Recommended storage conditions: The product is stable during short time storage between 2–40 °C (for example during transportation), but for long time storage it is recommended to store the product at room temperature 15–25 °C away from moisture, direct heat and direct light.
- SmartPAN must not be used after the expiry date.
- Do not use if the package is opened or damaged.
- Keep out of the reach and sight of children.
- Prescription only.
- Discard any opened or empty syringes as biohazard.

Dosage / administration

- The dosage of SmartPAN can vary, depending on the size of the tissue surface of interest. Standard applicator contains 10 ml of the product. This amount will allow for repeated application in one patient on standard pancreatic cut surface or anastomosis.
- SmartPAN is provided sterile, in a pre-filled, single-use applicator. It is ready-to-use and does not need any preparation.
- To prevent inadvertent flow of SmartPAN hydrogel to unintended areas, the area of interest should be maintained in a horizontal position and SmartPAN should be applied from above.

Reporting

All serious or life-threatening adverse events or deaths associated with use of SmartPAN should be reported to the

Competent Authority of the country where it occurred and to the device manufacturer at info@maglechemoswed.com

Date of information

2021-05-06, CC 2023

Intellectual property rights

Patent: SmartPAN technology is protected by patent rights. Visit the Magle Chemoswed website for further information.
Trademark: SmartPAN®

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10 ml

magle
chemoswed.

Luer The device has a Luer-Lock connection **Rx only** Prescription only

REF Catalogue number **40°C** Temperature limit

CE 2409

REF 493073

960740-01

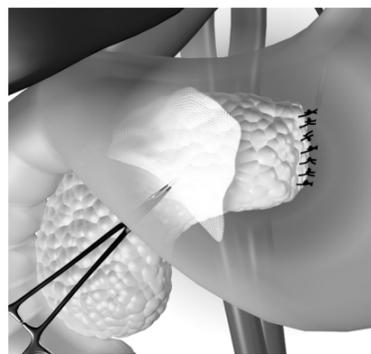
EN

1. Inspect that the packaging is intact. Remove the applicator from the blister package. Remove the cap.

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
2. When SmartPAN is used, standard surgical preparation at the pancreas should be finished such as organ exposure, partial removal, closure of the pancreatic remnant or pancreoenteric anastomosis. Additionally, standard hemostasis should be finished, and traces of blood should be carefully removed by rinsing and dabbing to prevent optical interference with the indicator reaction.

Moisture does not impair the function of SmartPAN. However, relevant fluid collections should be removed prior to usage in order to prevent dilution of the product.

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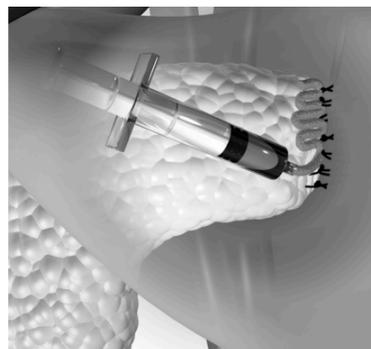
3. Move the applicator to the surgical field without unintentional placement of the SmartPAN hydrogel on the patient or surroundings. For open surgery, hold the applicator in your hand like a syringe using your thumb to push the plunger gently linearly along the inside of the barrel. With the applicator jet downward, slowly apply SmartPAN in one continuous layer to the tissue surface without local volume accumulation or time-delay between start and end of application.

For minimally invasive surgery, connect the applicator to the cannula. It is advised that a new type of cannula is always tested outside the patient before used in a patient for the first time to confirm that SmartPAN flows easily through the cannula.

Move the applicator or cannula back and forth over the tissue in an s-shape while applying the gel continuously. To provide optimal visualization of pancreatic leakage, SmartPAN should completely and homogeneously cover tissue surface, and time-delay between start and end of application should be avoided.

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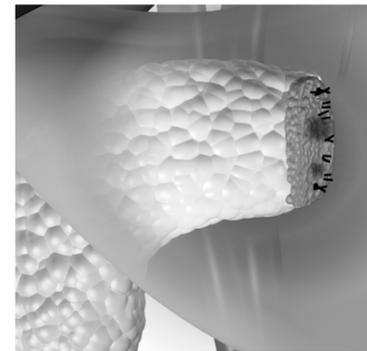EN

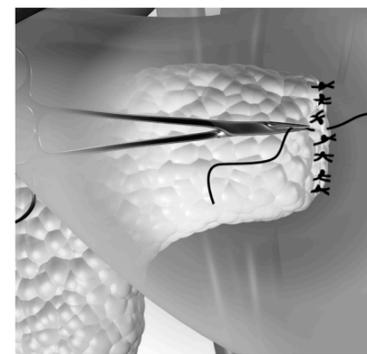
4. SmartPAN appears orange when applied, but locally turns green-bluish over pancreatic leakage. This indicator reaction happens immediately or within seconds. Color change only appears in close proximity to relevant pancreatic leakage.

It is recommended that a negative indication is followed by rinsing the product with sterile isotone saline and careful dabbing of the tissue surface and re-application of a second layer for verification of negative indication.

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

5. When pancreatic leakage becomes visible, targeted closure using surgical means may be performed. SmartPAN does not need to be rinsed of before targeted mechanical closure (such as sutures or surgical clips) unless it does hinder surgical procedures. For chemical/electrical targeted closure (such as tissue glue, tape or non-mechanical tissue sealer) it is recommended that SmartPAN is rinsed prior to targeted closure to allow proper closure.

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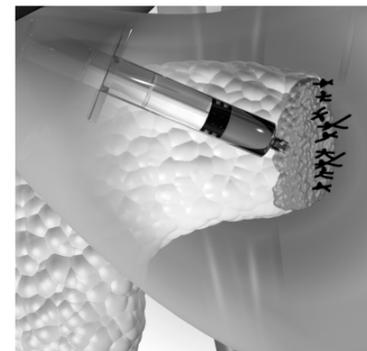6. It is recommended that SmartPAN is re-applied after targeted closure to confirm tightness of the action.

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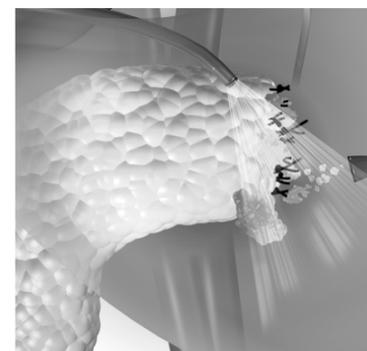7. After usage, rinsing of the surgical site and drainage or suctioning of jet fluid should be assured to avoid accumulation of the product in the abdominal cavity. For minimally invasive surgery it is advised to use flushing combined with parallel suctioning of jet fluids to avoid retention of jet fluids.

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